

Bruce Bed assembly instructions: Paul Weiss, SMRRC 7/21/2010

1. Roll the rack to bed assemble location.



2. Lock rack wheels by stepping on wheel lock lever.



3. Remove Nylon cover (velcro) on the open (non barred) side of the rack.



4. Disconnect the 2 straps that hold the beds on the rack. These are the vertical straps. The buckles are near the bottom of the rack.



5. Remove beds and lay on floor.



6. Remove Clevis pins (4) from leg assemble. These are attached to the bed so they cannot get lost. The clevis pins have their own locking mechanism. They are

easy to remove.



7. Pull leg assemble out from bed.



8. Straighten legs.



9. Push legs back into the same bed holes you just removed from. This time put the legs through the extra braces. Red arrow should point towards each other.



10. Put the 4 clevis pins back through holes to lock the legs in place.



11. Lift bed into normal upright position. You may lift the head of the bed upright if you want.



12. Remove nylon Oxygen bottle holder which is on top of the bed and attach to the back of the bed on the headrest end.



13. Detach the IV pole, which is tied underneath the bed and attach to the side of bed (either side) near the patients arm. It loops behind the top bar and over the bottom bar.



14. The finished bed can be moved on its caster wheels, but this bed is **not meant to be moved with a patient on it**. You should lock the wheels once bed is in place.



15. Beds also have a Triage Tag color system. Each color stick can be raised, based on patient emergency triage status.



Instruction video available at:

<http://www.smrrc.org/Videos/Bruce%20Bed%20Video-H.264%20800Kbps%20Streaming.mov>