

Quick Reference Guide

Sure Clean HFS Portable Shower

Description:

The Sure Clean HFS Portable Shower is a heavy duty two panel collapsible lightweight aluminum shower that telescopes to a sturdy full size shower when deployed. It is designed for repetitive use to decontaminate individuals following a hazmat or mass casualty incident. The unit has:

1. A shower head with a 5 foot reinforced hose.
2. A removable curtain.
3. A quick release pin system.
4. A removable bottom basin.
5. Heavy duty carrying handles.

Assembly:

The unit can be assembled by one person. Pull the pins located on the side walls. They are spring loaded and will need to be held open in order to raise the top portion of the shower. Place one hand in the center of the shower and raise as you pull pins one at a time. When the shower is up, verify that the pins have locked into the receiving holes so the shower does not drop onto the patient. Assemble shower curtains. Attach water supply hose to the top side of the shower. Attach drain hose to the bottom of the shower and lead to appropriate drainage area.

Maintenance:

The shower can be easily moved for decontamination and should be decontaminated per hospital protocol.

Other:

The patient should complete the Pre-Decontamination Procedure prior to entering the shower.

This guide is intended as a quick reference only. Receivers/users assume all responsibility for the proper maintenance and use of this equipment. This guide is not intended as a substitute for manufacturer provided reference materials and user instruction. This guide is available upon request from SMRRC.